
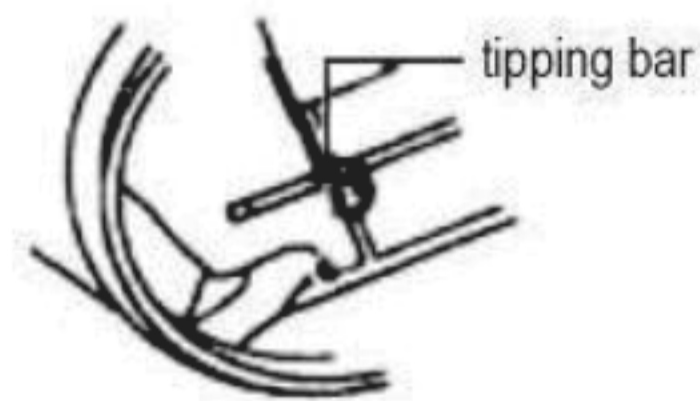



Wheelchair user guidance

4.

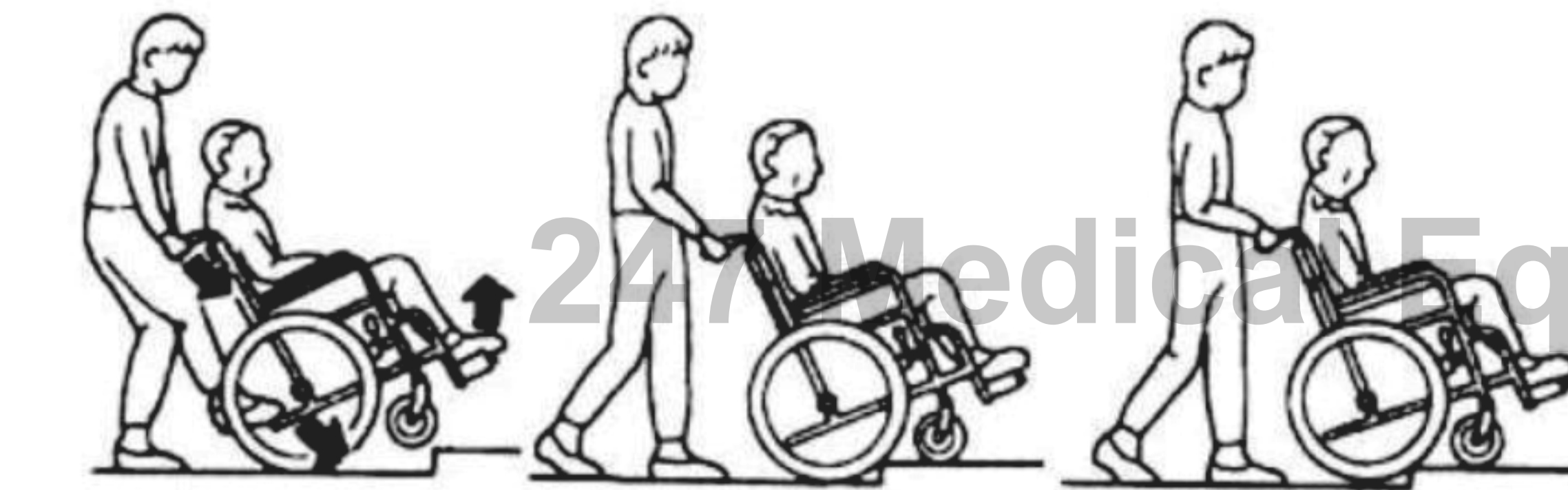

To move up a step, the care giver pushing the wheelchair must step on the tipping bar and push downwards at the handles such that the front wheel castors are raised onto the step. Next pull upwards at the handles to bring up the rear wheels. Do not try to push over the steps by force.

5.

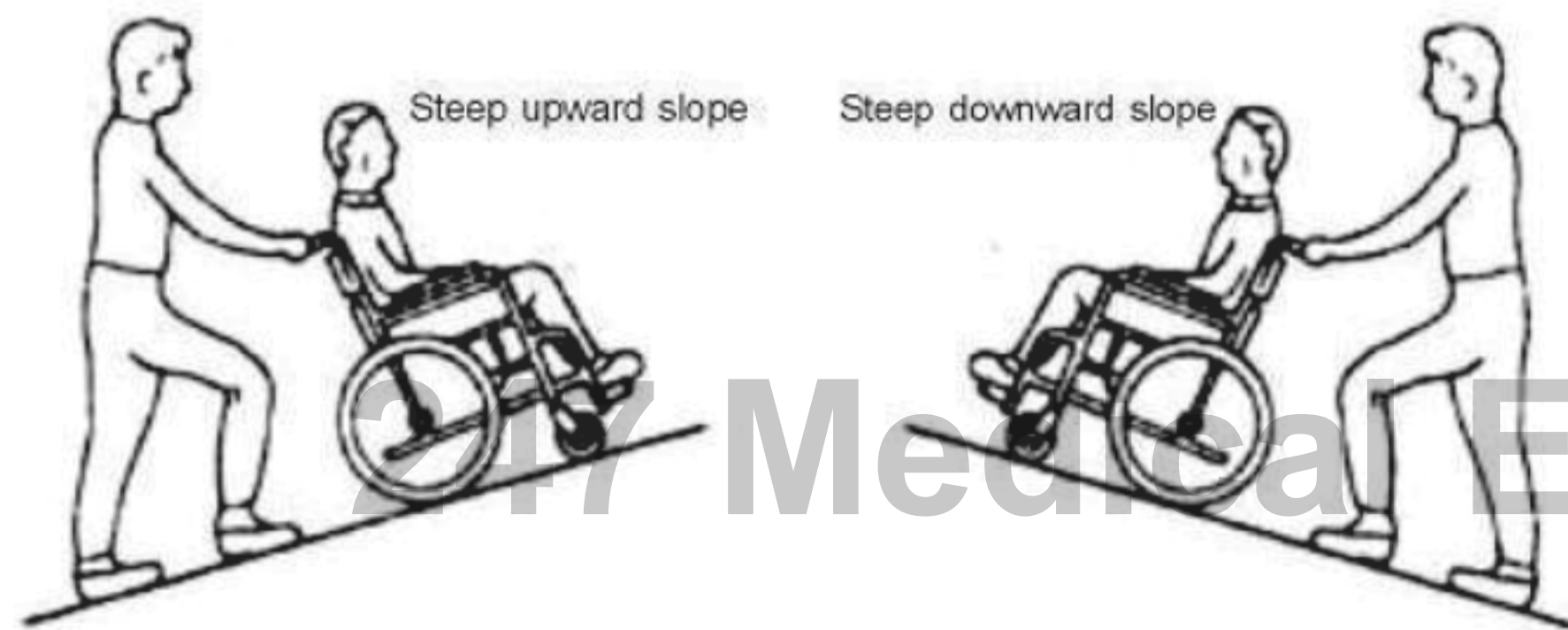

To move up or down a steep slope, ensure that a care giver is around to support the wheelchair, otherwise, the chair may overturn. For a steep downward slope, the wheelchair should be turned to face up slope, and the care giver should back the chair downwards gradually.

6.


When lifting a wheelchair with the patient on it, do not hold on to the handles if the chair's backrest is the foldable kind. Otherwise, the backrest may be damaged.




247 Medical Equipment LLC - Dubai For manual wheelchairs

1.Parts of the Wheelchair

SELF MANEUVERED

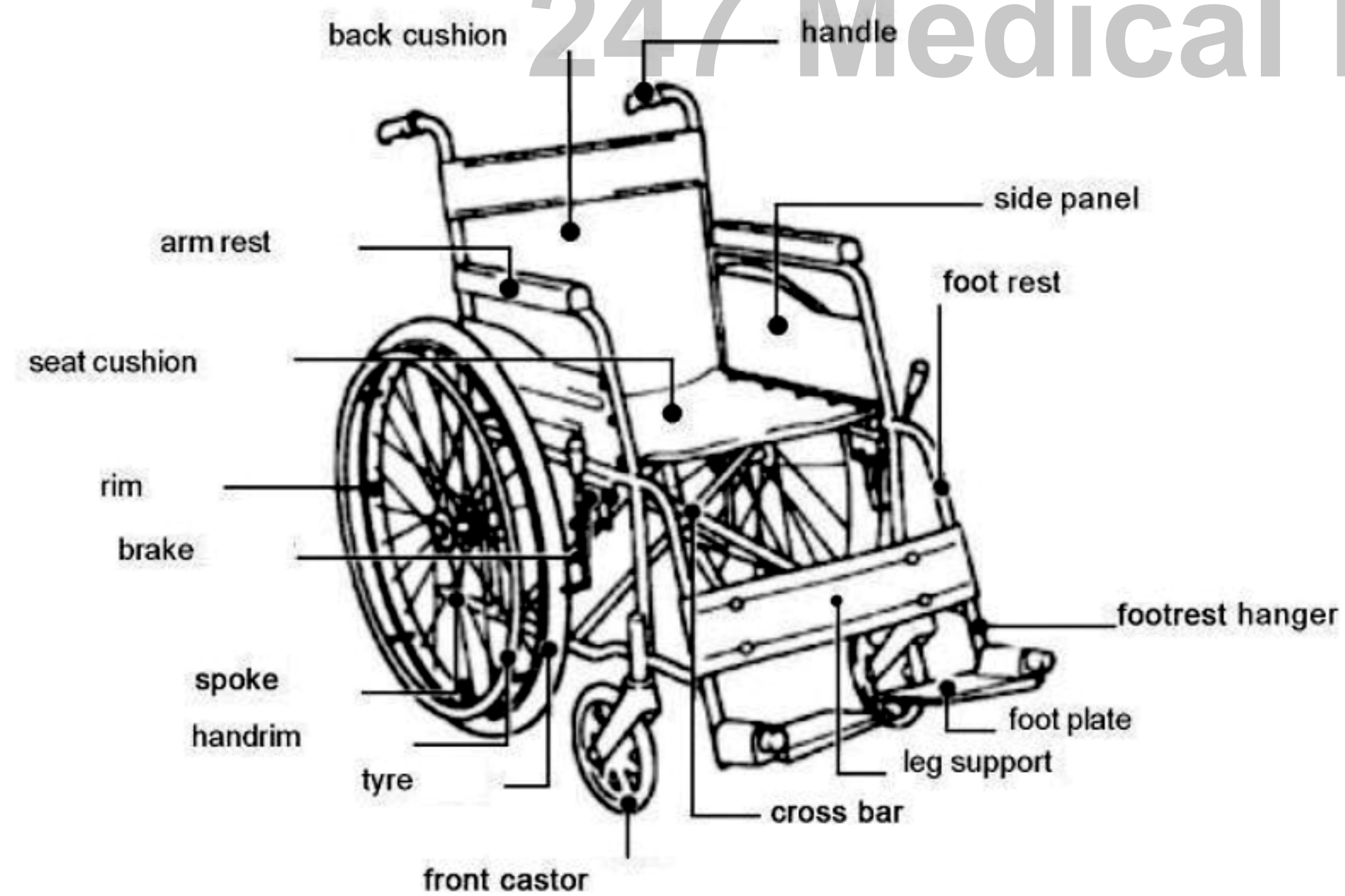


PATIENT ASSISTED

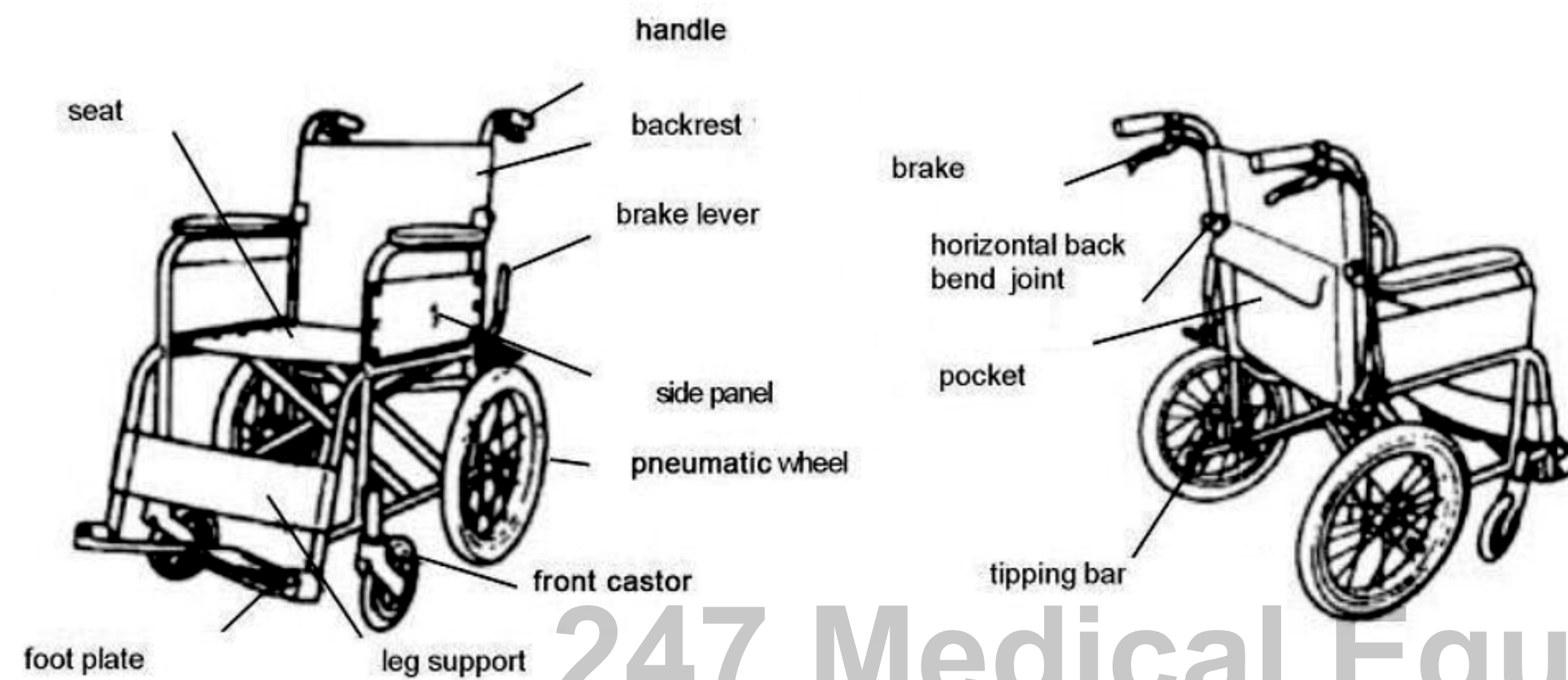


4.To Note When Using the Wheelchair


1. Ensure that the brake is locked when getting on/off the wheelchair. Refer to page 6 on how to use the brake

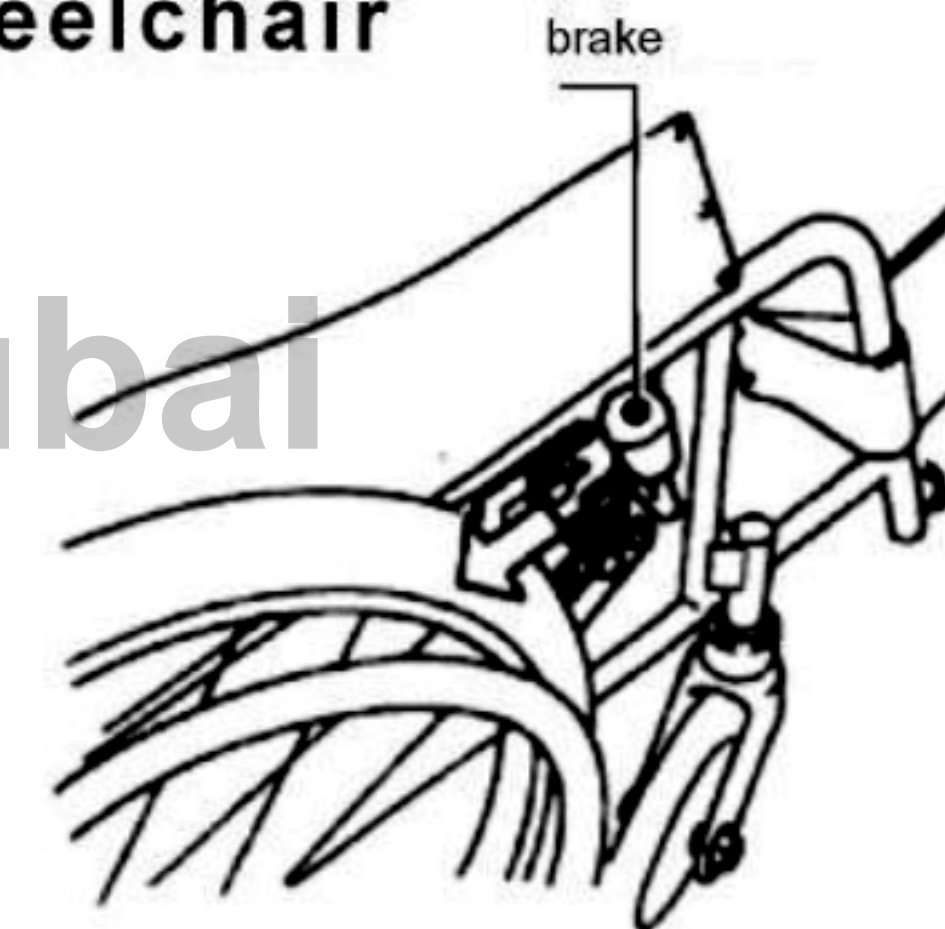

2. Do not step on the foot plate when getting off the wheelchair, or else the chair will tip forward.

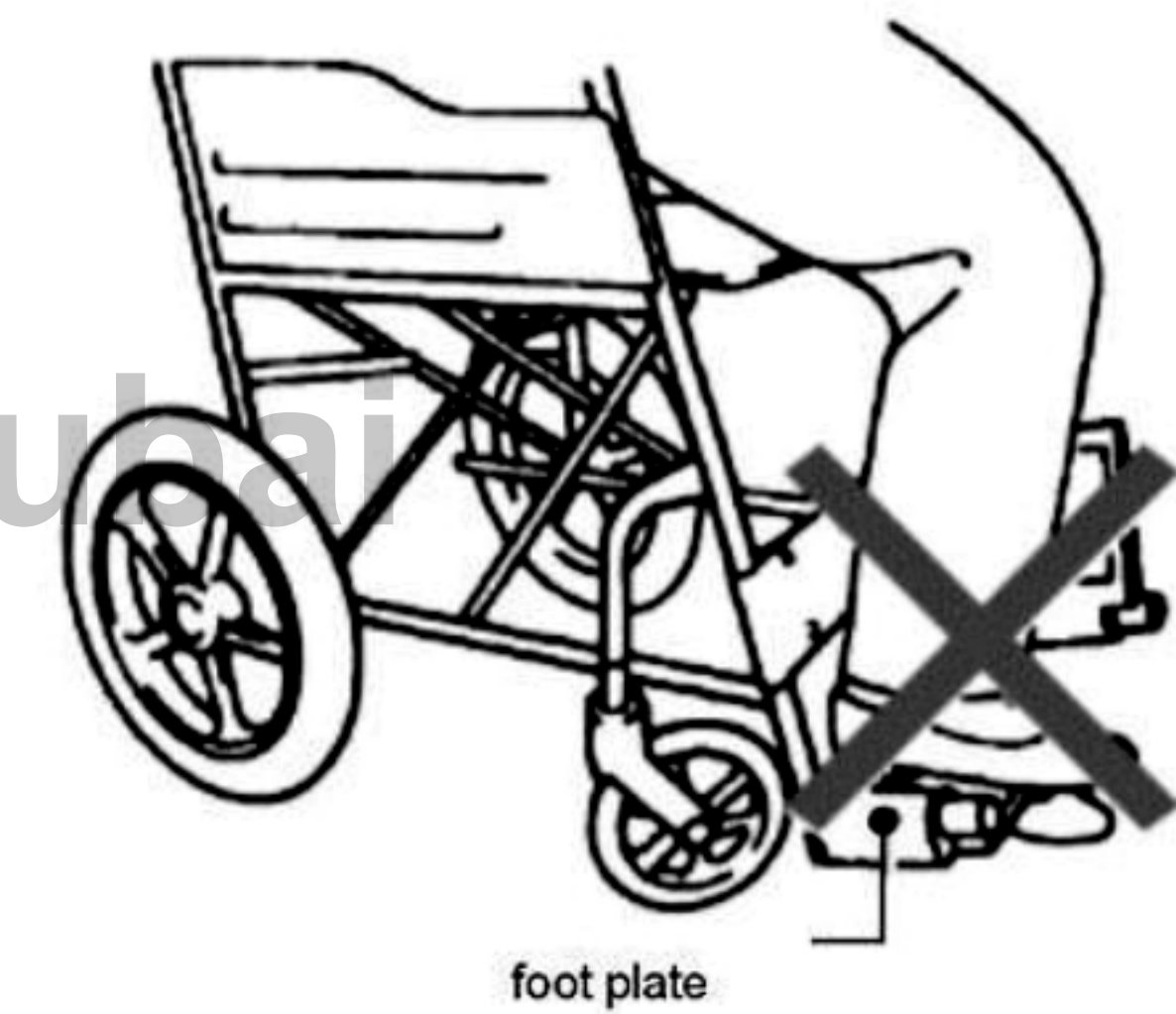


3. Do not place the wheelchair near any sources of fire, or else the backrest may be set on fire, resulting in injuries.

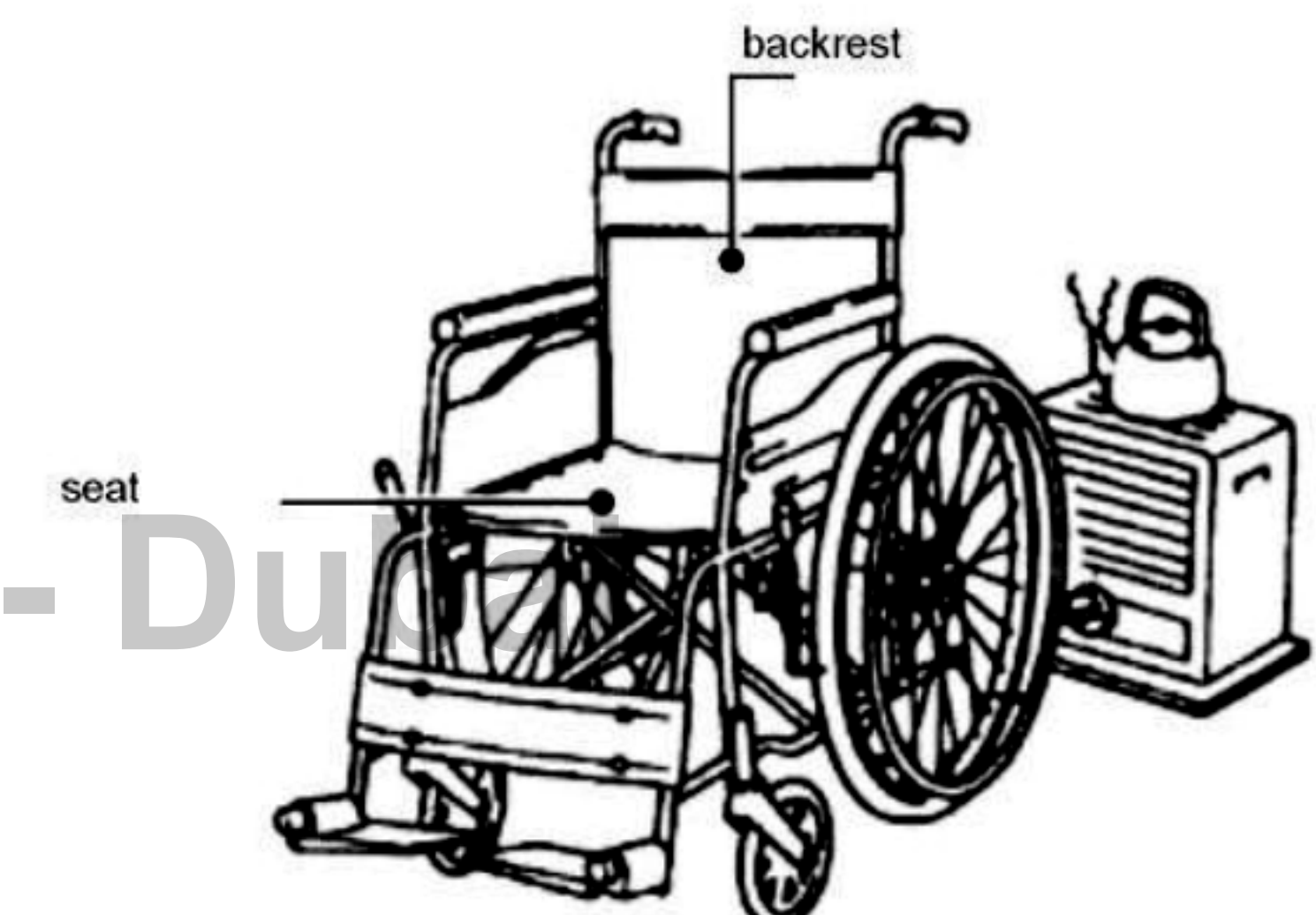


USING THE BRAKE

1. For Self Maneuvered Wheelchair

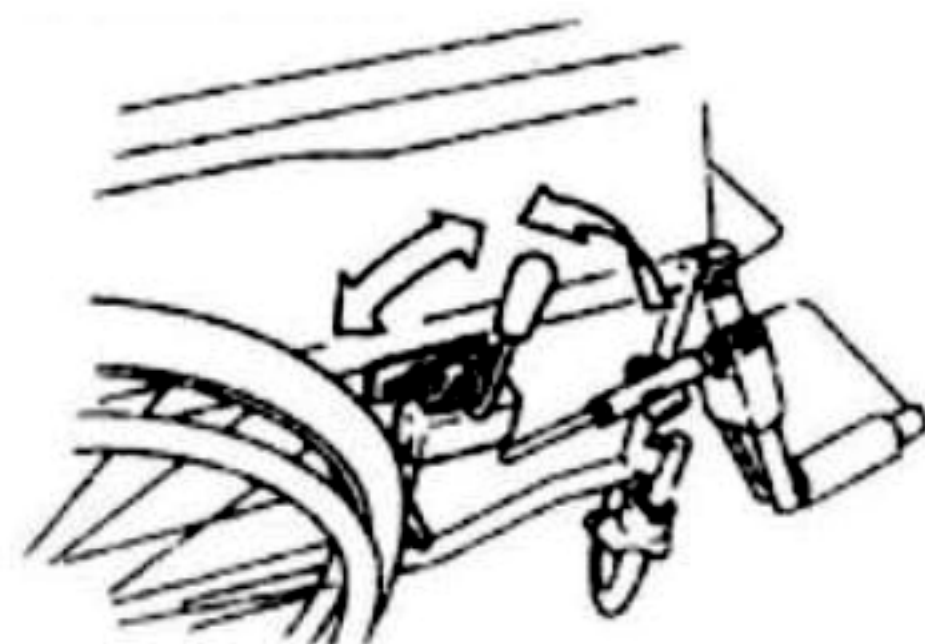


Pull both brake lever backwards to lock chair.
To release the brake, push it forward.

2. For Patient Assisted Wheelchair

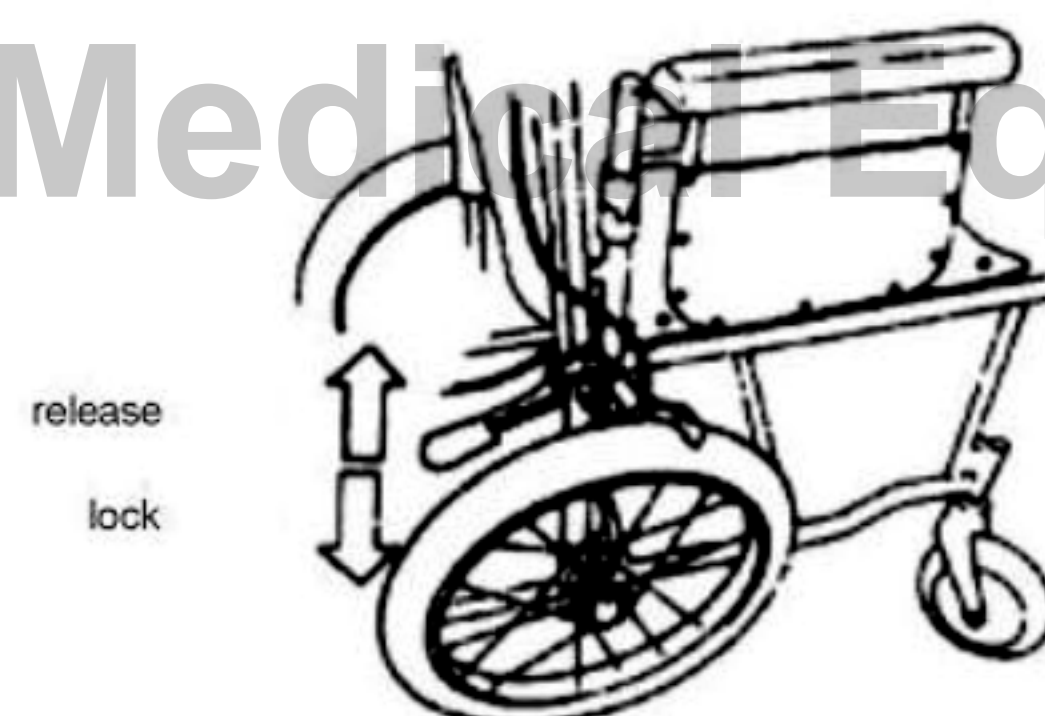

Push both the brake lever downwards to lock the wheelchair, and pull the lever up to release the brake.

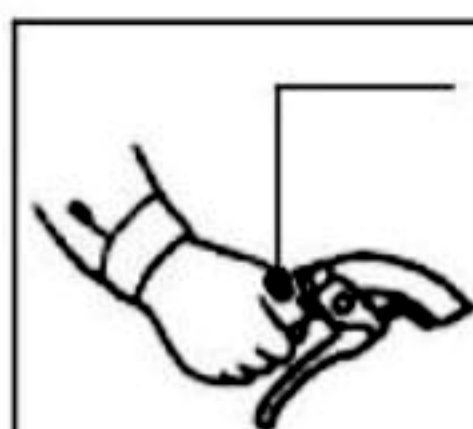


Brakes at the handles are only for reducing speed. Remember to apply both the brakes simultaneously. To lock the chair, the brake lever must be applied.

CAUTION

1. Do remember to engage the brake even when stopping for a short while, make it a habit to engage brakes.
2. The wheelchair is unable to execute sudden stops, so do not speed while on it.
3. When turning at a sharp corner, reduce speed to avoid tipping over.
4. Before moving the wheelchair, ensure that the patient has his/her feet on the foot plate.
5. Always ensure safety first. If the patient feels unsafe being alone, he/she should ask for assistance before proceeding.

1. In Folded/Packed Condition

With Fixed Backrest

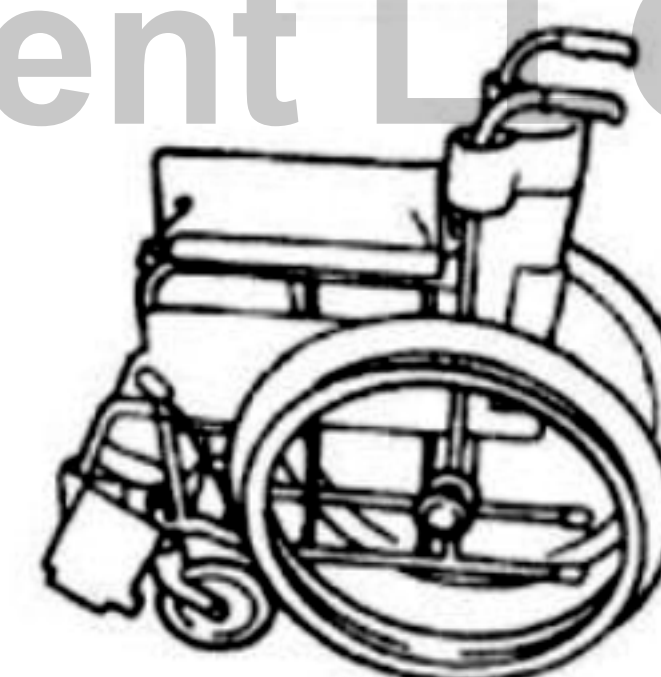

With Foldable Backrest



ACCESSORIES

1 pc Spanner


WHEEL CHAIRS



Operation Manual


OPENING THE WHEELCHAIR

CAUTION

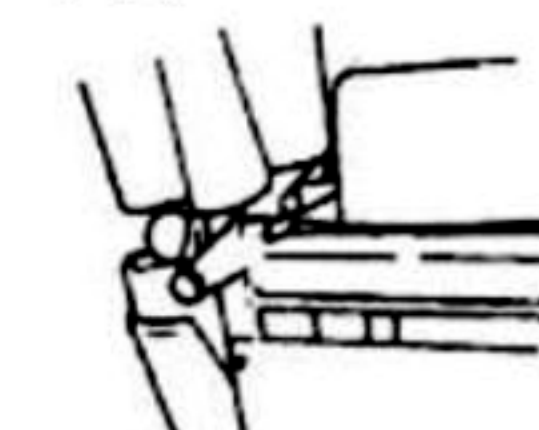

When the pin is locked, a click sound can be heard.

With Foldable Backrest

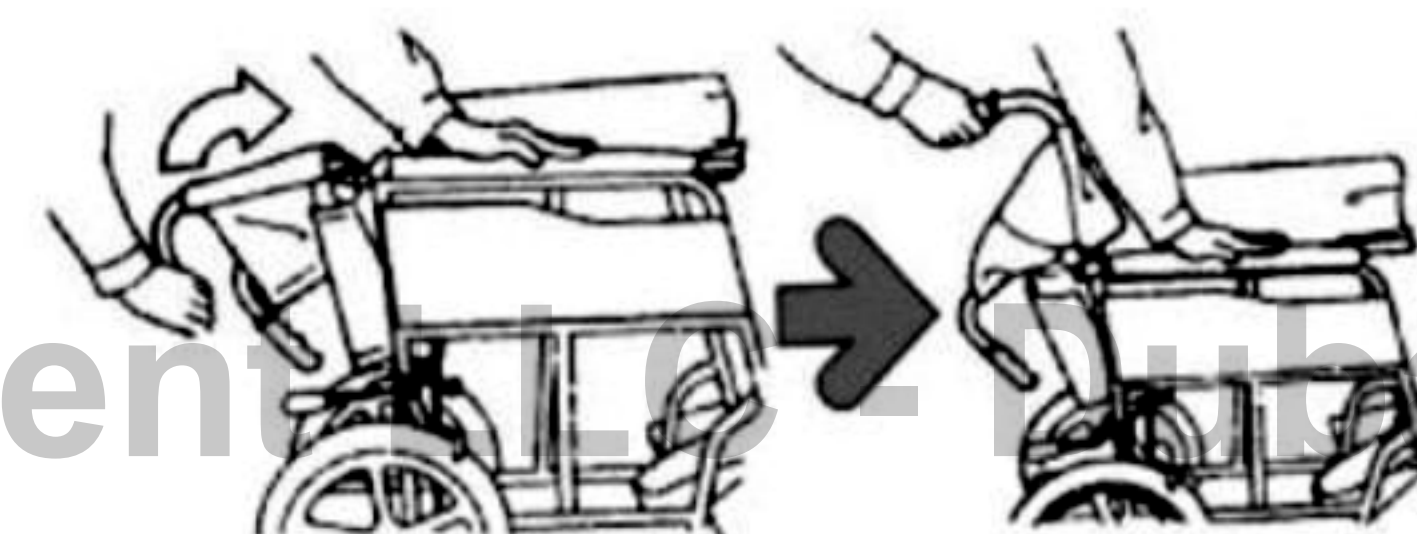

1. Grasp the handle and lift upwards in the arrow's direction.
2. While lifting the handle, push against the armrest for support.

With Fixed Backrest



Stand behind the wheelchair and pull it outwards to open up the chair.

Next press the seat down to open it.

CAUTION



Seat pipe 座位支撑管


Do not place your hand or finger at the seat pipe or beneath the seat.

Adjusting the Height of the Foot Plate



Turn the foot plate in the direction of the arrow, and adjust its height. Then tighten the nut at the side of the step pipe with the spanner provided.

Adjusting the Height of the Foot Plate

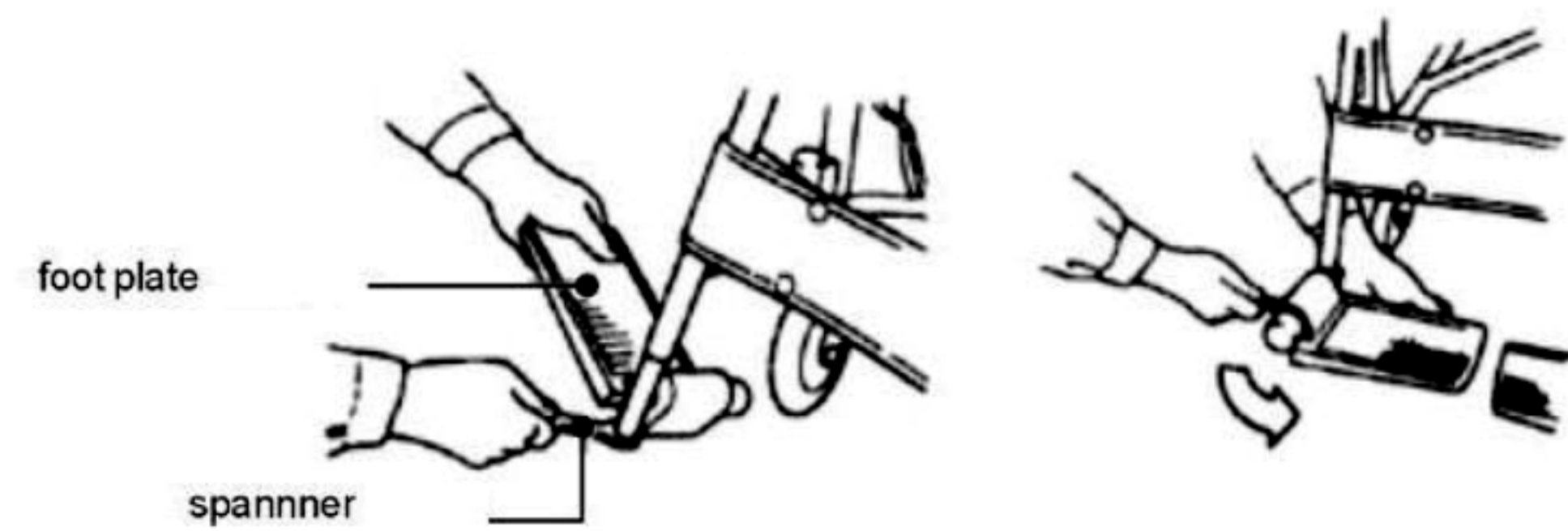


1. Using the spanner, slightly loosen the portion of the step pipe where it is bolted to the foot plate.

2. Turn the foot plate in the direction of the arrow. Then fix the foot plate.

FOLDING UP THE WHEELCHAIR



Fixed and Foldable Backrest

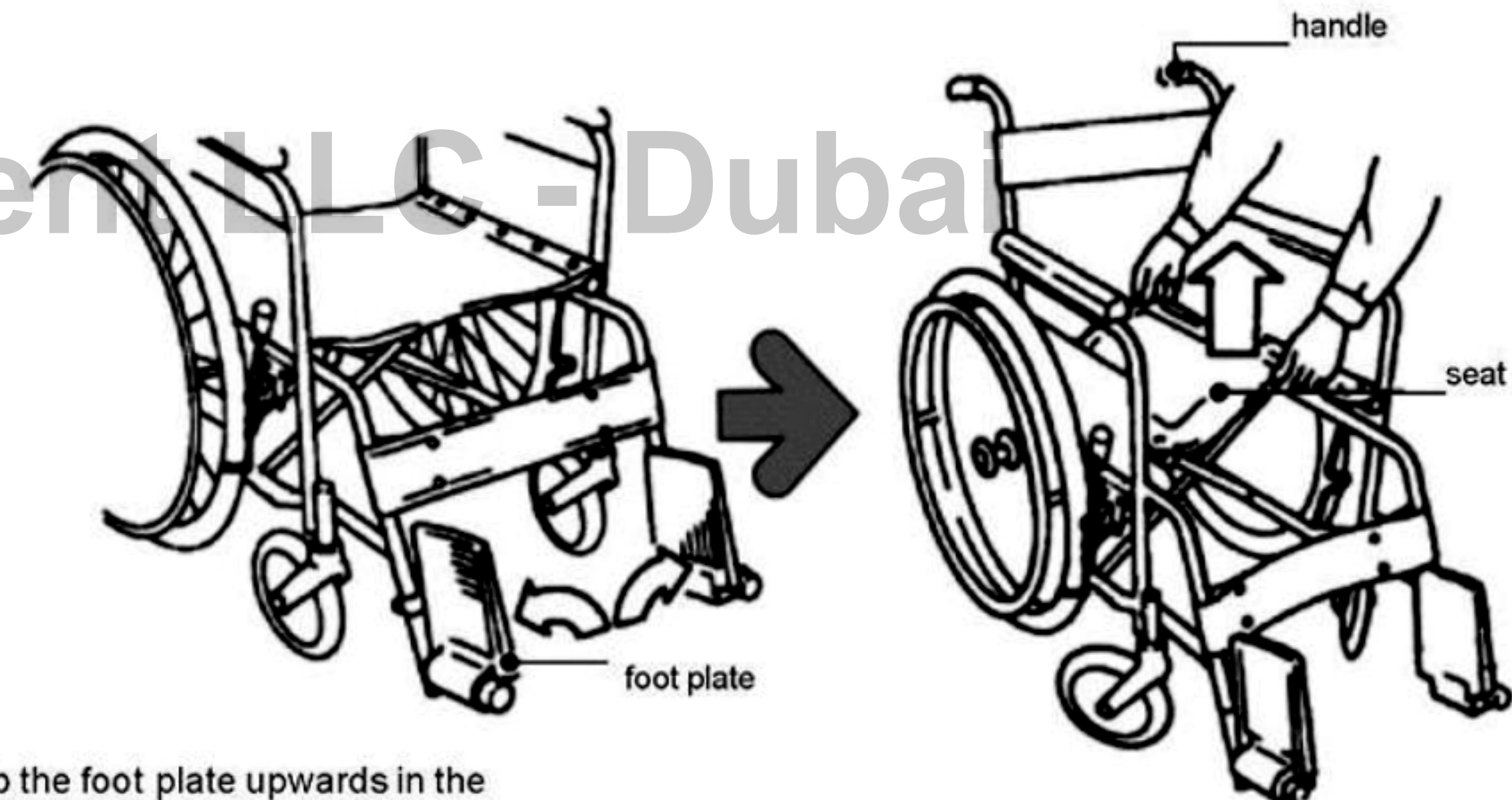

1. Flip the foot plate upwards in the direction of the arrows.

2. Lift the centre of the seat up to fold the wheelchair.

Steps ①&② are applicable for wheel chairs with fixed backrests. For those, with foldable backrests, continue with steps ③&④.

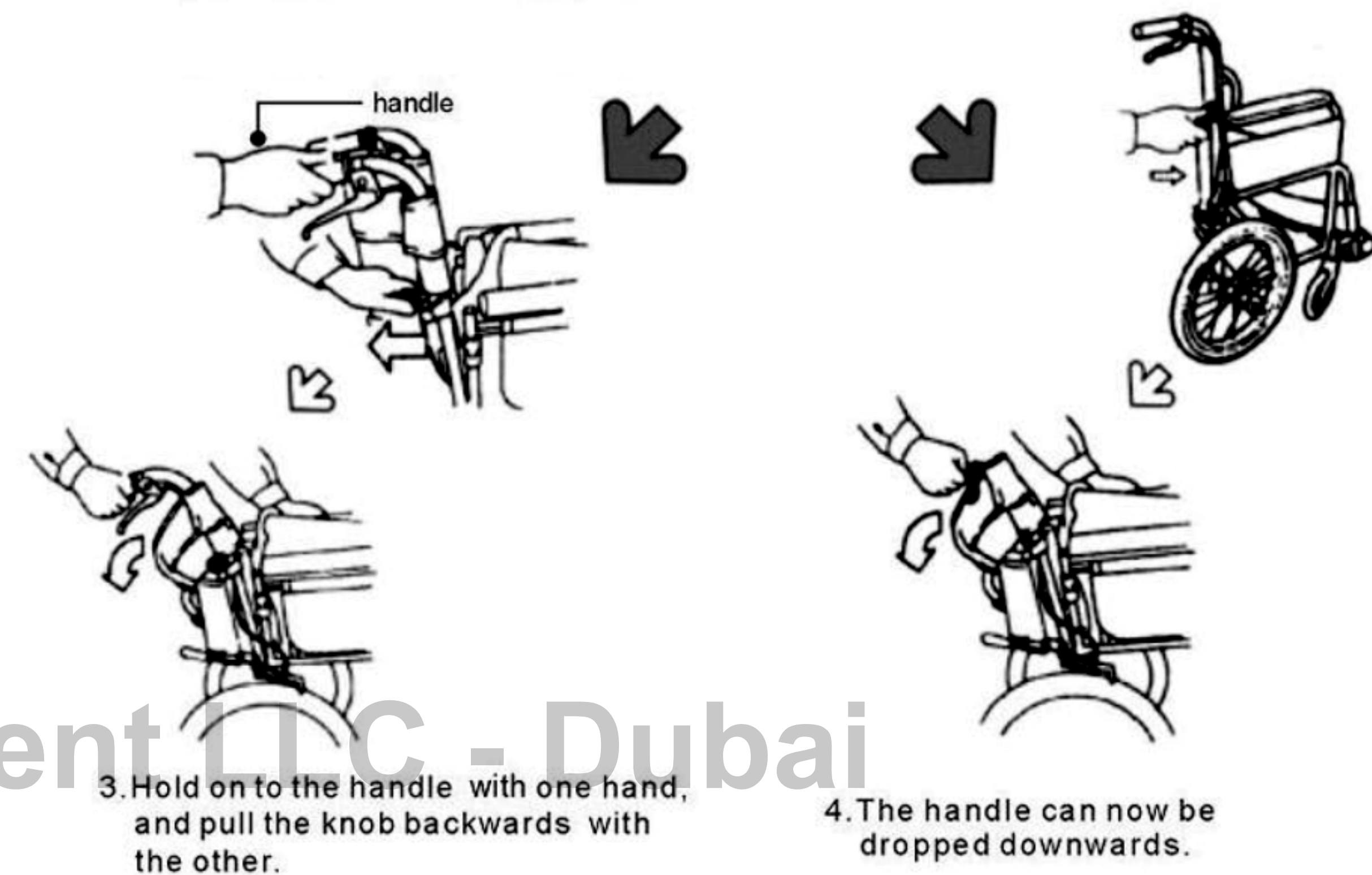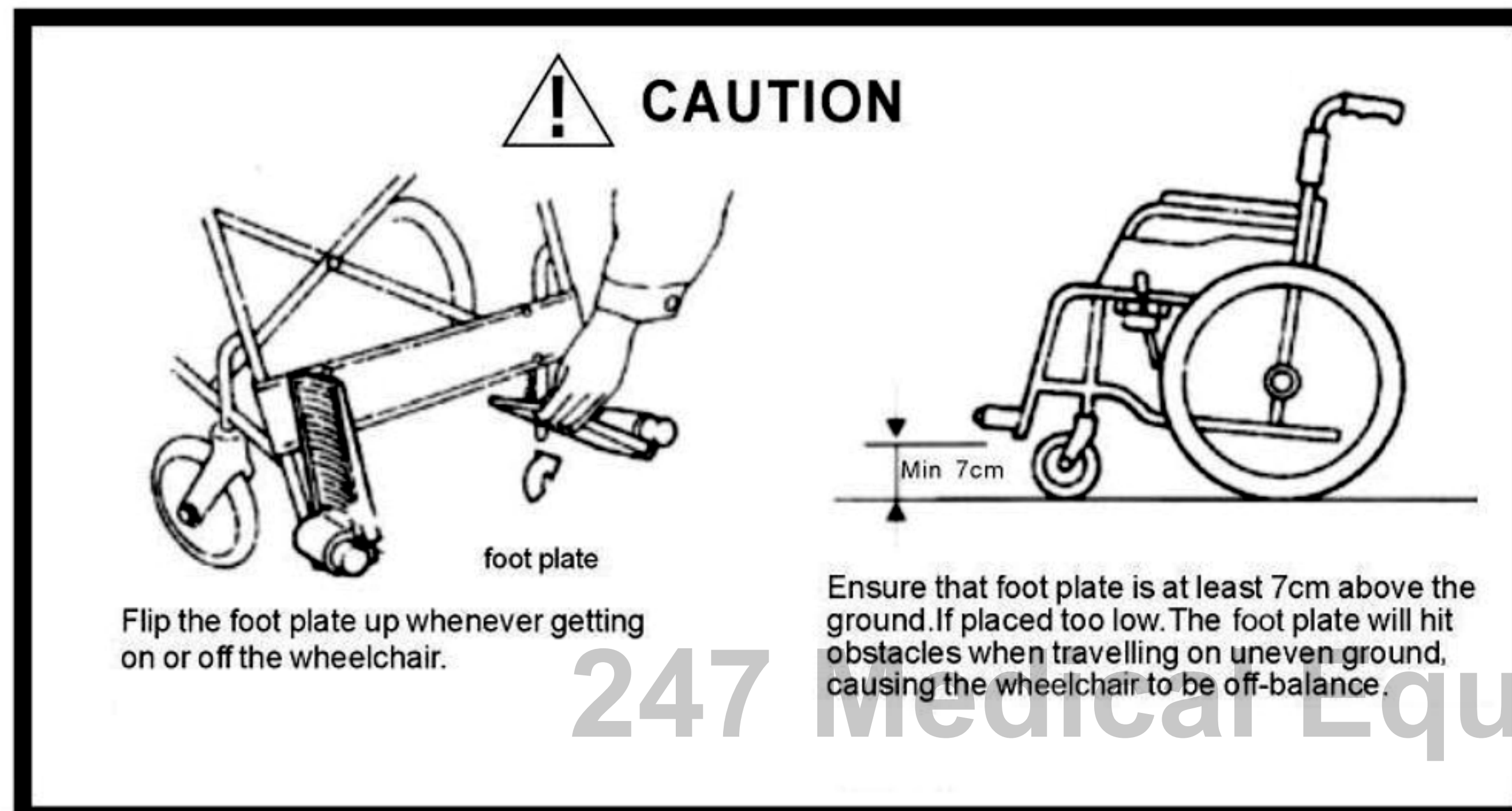

3. Hold on to the handle with one hand, and pull the knob backwards with the other.

4. The handle can now be dropped downwards.